

Medical Protection Cover

Ultrasound Probe Cover SGMPC1530A/SGMPC14240/CUSTOMIZED

PURNOTE protection cover - C arm/C-arch protection cover is an envelop made of thermoplastic polyurethane. It can offer the probe an isolated condition from the patients. As a result, the cover has a good capacity of waterproof, anti-bacteria and anti-pollution. It can save the cost and time of the machine infection and procedure turnover.

Intended application

For Ultrasound probe protection


Material

Thermoplastic polyurethane




Warnings


Don't use if package is damage

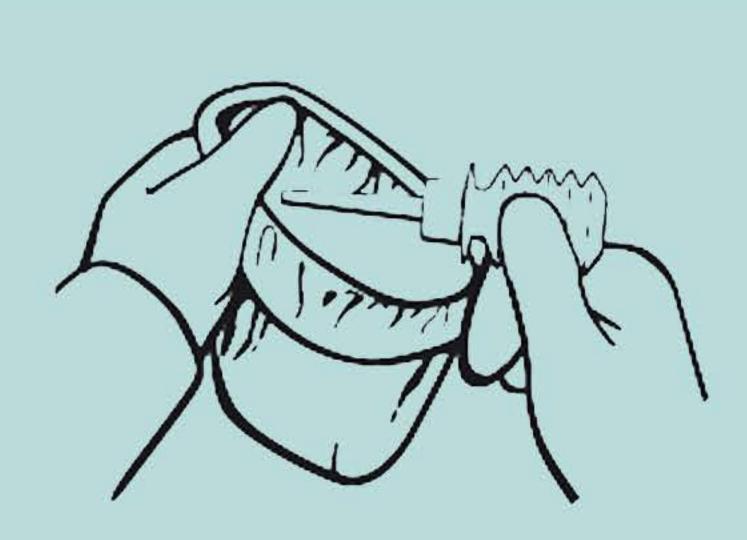

1. Place a sufficient amount of ultrasound gel in the cover and / or on the surface of the transducer. If no gel is used, the quality of the image can suffer.

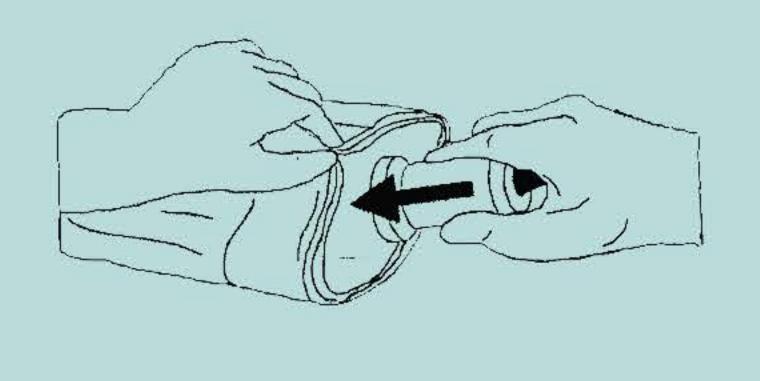


2. Insert the transducer into the cover. A suitable sterile method must be used for this. Pull the cover tightly across the face of the transducer to remove wrinkles and air bubbles. Make sure that the cover is not punctured in the process.

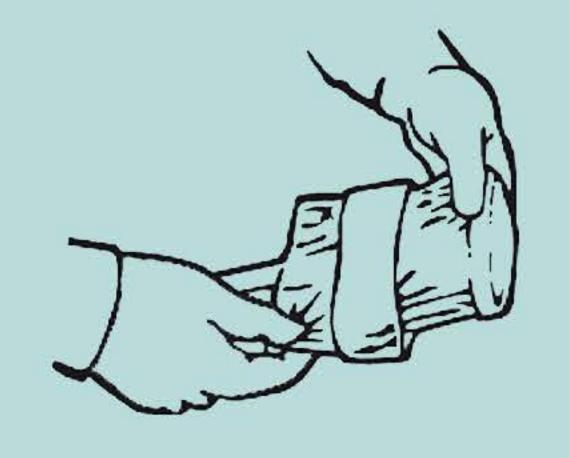

3. Place the cover tightly over the transducer. Push away any air bubbles with your fingers.

Grip the open end of the cover and

Grip the open end of the cover and simultaneously hold the wrapped transducer firmly in your hand. Then pull out the cover along the cable. Do this in two steps in the case of longer covers.

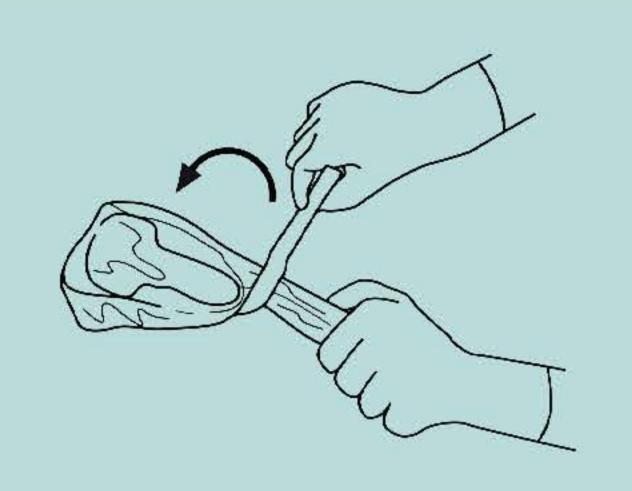

4.Attach the cover with the adhesive strips or rubber bands provided.








Ultrasound Probe Cover

Product No.	Size	Packing
SGMPC1530A	14*30cm/5.9"*12"	Single wrapped,100pcs/box;400pcs/carton
SGMPC1450A	14*50cm/5.9"*20"	Single wrapped,100pcs/box;400pcs/carton
SGMPC1490A	14*90cm/5.9"*36"	Single wrapped,100pcs/box;400pcs/carton
SGMPC14120	14*120cm/5.9"*48"	Single wrapped,100pcs/box;400pcs/carton
SGMPC14150	14*150cm/5.9"*60"	Single wrapped,100pcs/box;400pcs/carton
SGMPC14240	14*240cm/5.9"*96"	Single wrapped,100pcs/box;400pcs/carton

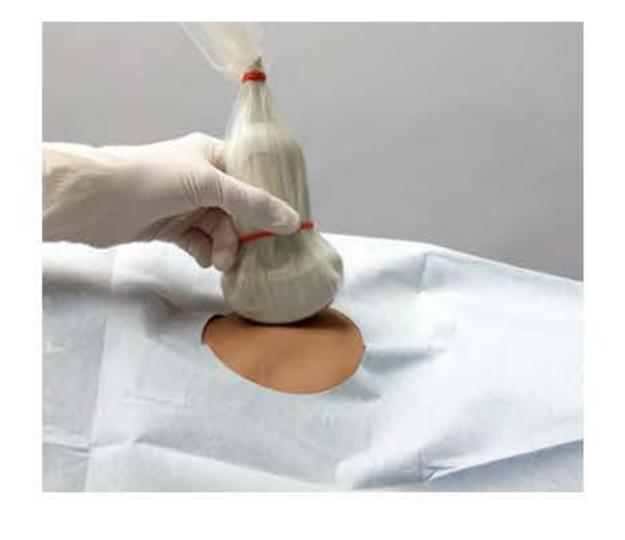



NANJING SENGONG BIOTECHNOLOGY COMPANY LIMITED

#201, Building 8, No.9 Kechuang Avenue, Jiangbei New Avenue, Nanjing City, 210000, China www.purnote.com Grel_xu@purnote.com Whatsapp&Wechat: +86 19962070258