


# Detectable Positioning Tape



PURNOTE

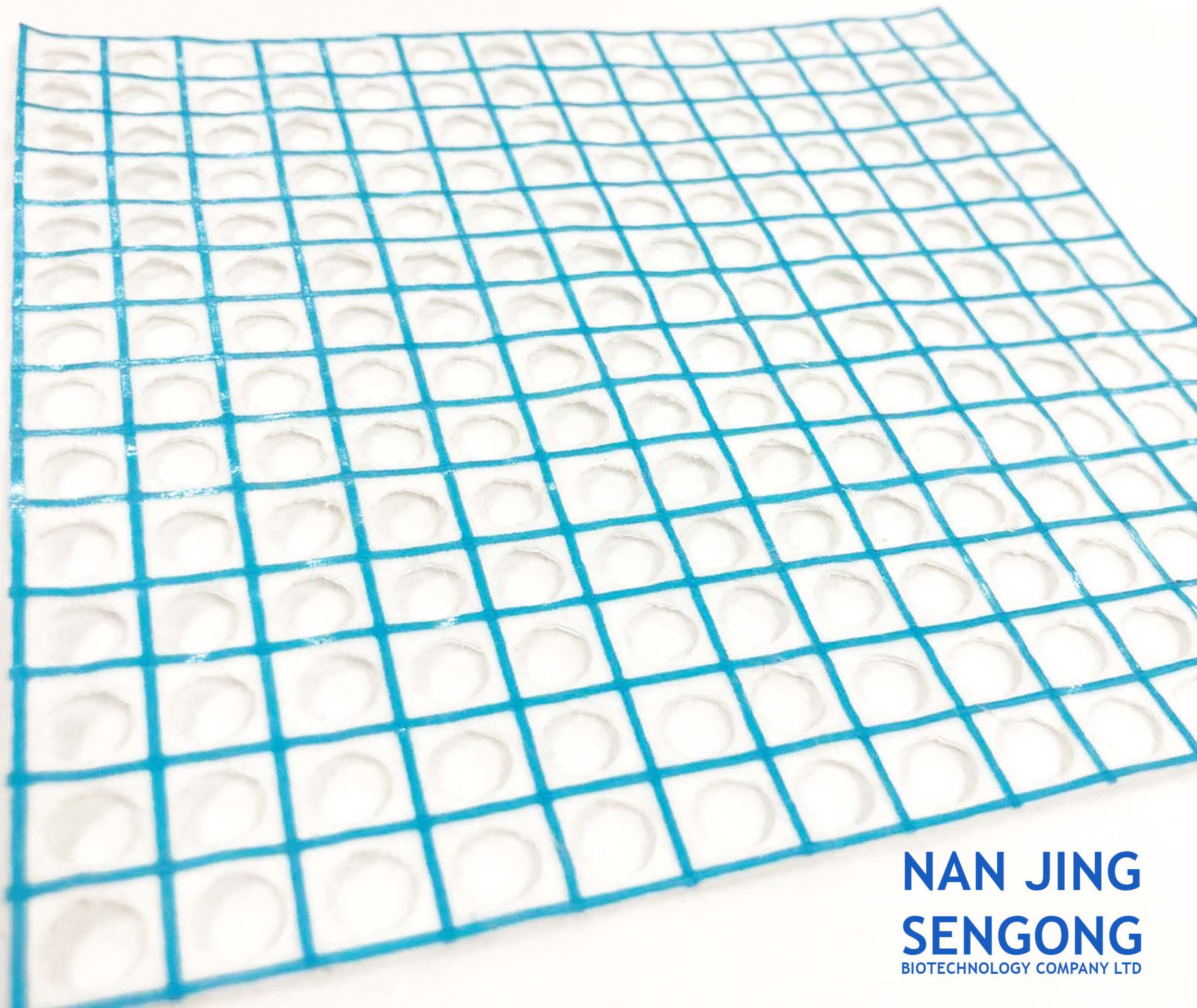

**NAN JING  
SENGONG**  
BIOTECHNOLOGY COMPANY LTD

# Detectable Positioning Tape

# Surgical Waffle@

**Structure:** This product is made of Spunlace defreezing cotton, detectable mesh and medical adhesive tape. The front side of the cotton sheet is composed of X-ray developing mesh and the other side is treated with adhesive sticker.

**Intended use:** For CT-guided percutaneous puncture orientation. It can make the orientation process of percutaneous puncture guided by CT more fast, convenient and accurate. And the surgical waffle is easy to operate, and lower the rate of cross-infection in medical environment. It does play a key role in locating the lesions in vivo and selection for the best puncture point.


**Specification:** S (1-5 cm), M (6-15 cm), L (16-25 cm), XL (25-40 cm). Other details seen in the technical requirement.

## Application method:


1. The part of skin which will be punctured have to be disinfected before use the Surgical waffle.
2. Open the sterilized package and take out the Surgical waffle. Peel the back tape, and paste it on the part that needs to be located.
3. Scan the site to be punctured by CT, including the upper and lower levels of the lesion location.
4. Measure the distance between the puncture point and the lesion location.
5. Using the Surgical waffle to confirm the puncture and start the puncture.

## Recommended departments:

1. Intervention department
2. Respiratory medicine department
3. Oncology department
4. Orthopedics department
5. Pain treatment department

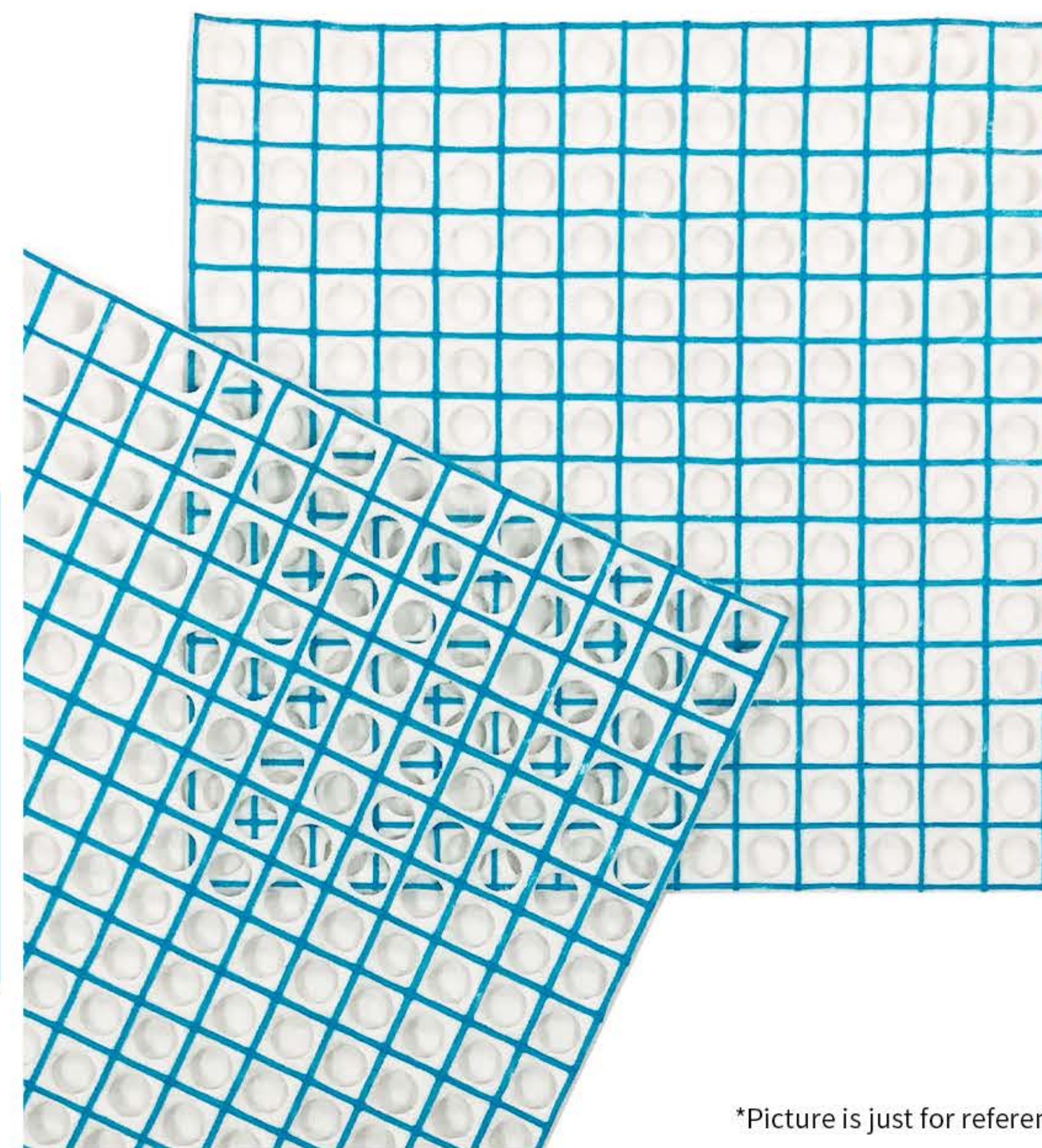


\*Picture is just for reference


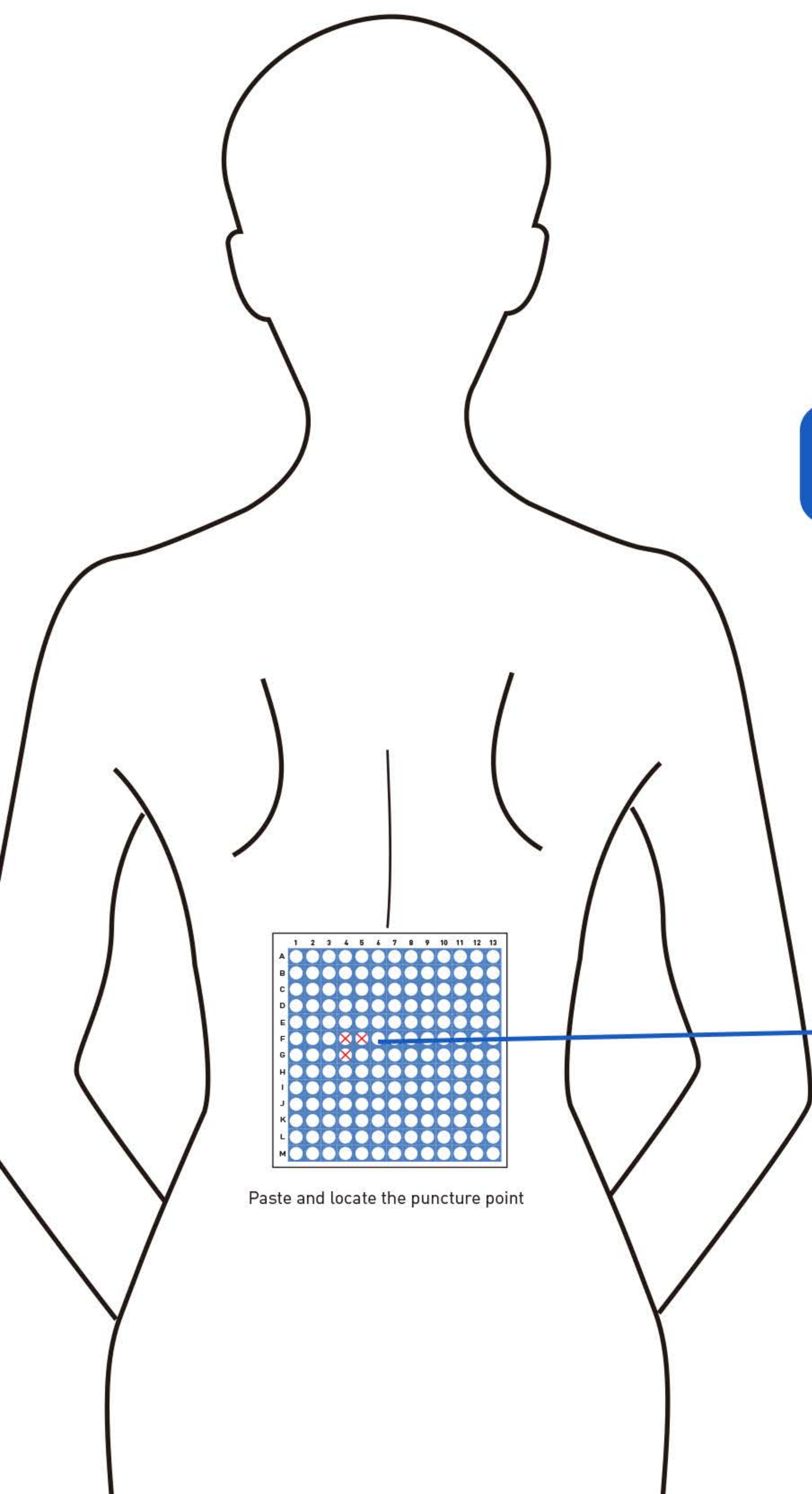


Application

Puncture

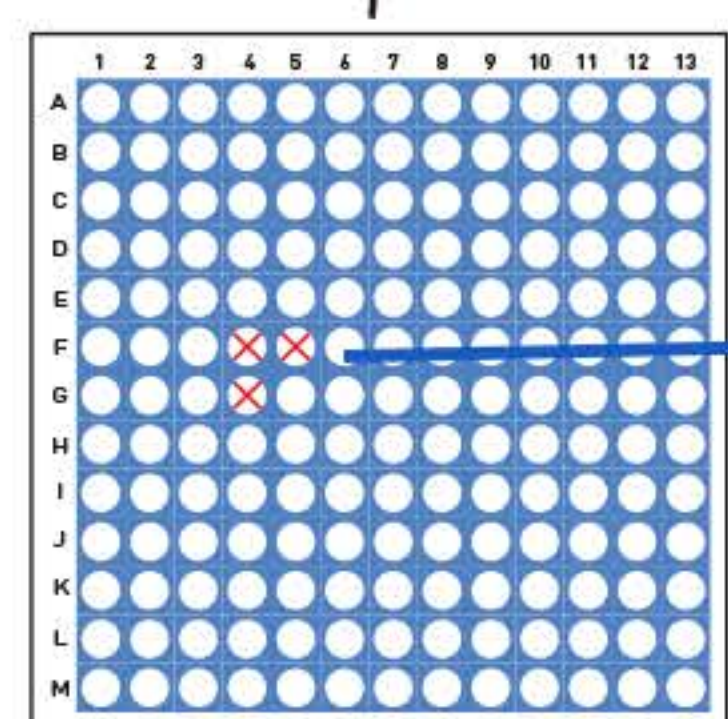

Paste and locate the puncture point

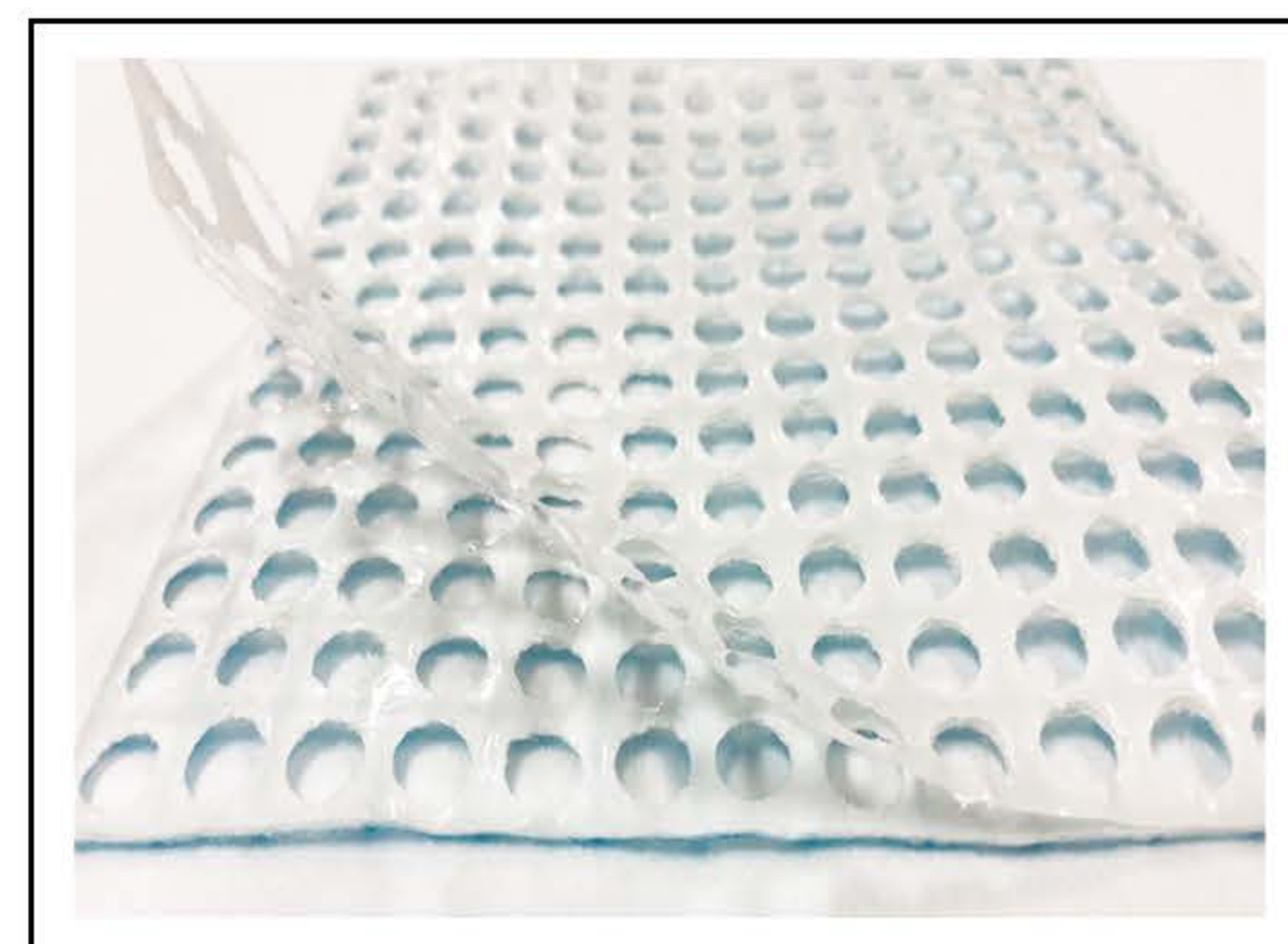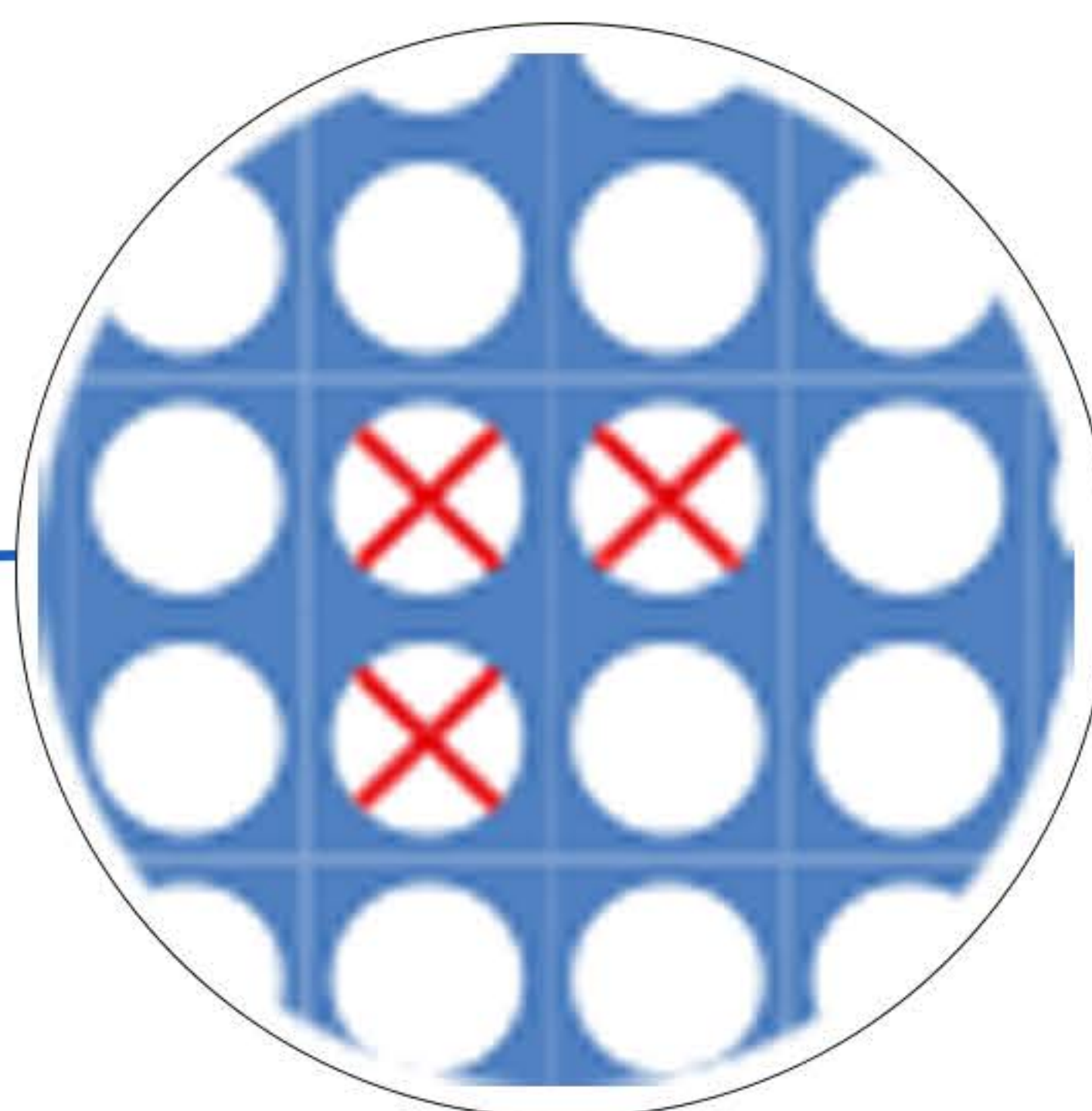

Medical-level tape



Effect display under CT monitor

# X-ray Detectable Cotton Sheet



# X-ray Detectable Cotton Sheet





The detectable cotton sheet is based on the X-ray detectable thread developed by our company completely and independently, and is formed by the ironing process. The use of this product resolves possible medical errors in the operating room.

Detectable thread supports customized development line with different tensile strength according to customer requirements, suitable for 0.2mm-2mm different types of subsequent processes.

Including cotton sheet ironing part and non ironing part can be developed. The cotton sheet is made of cotton spunlace cotton, which has good liquid absorption capacity, less wool and smooth cutting edge.



## Display:

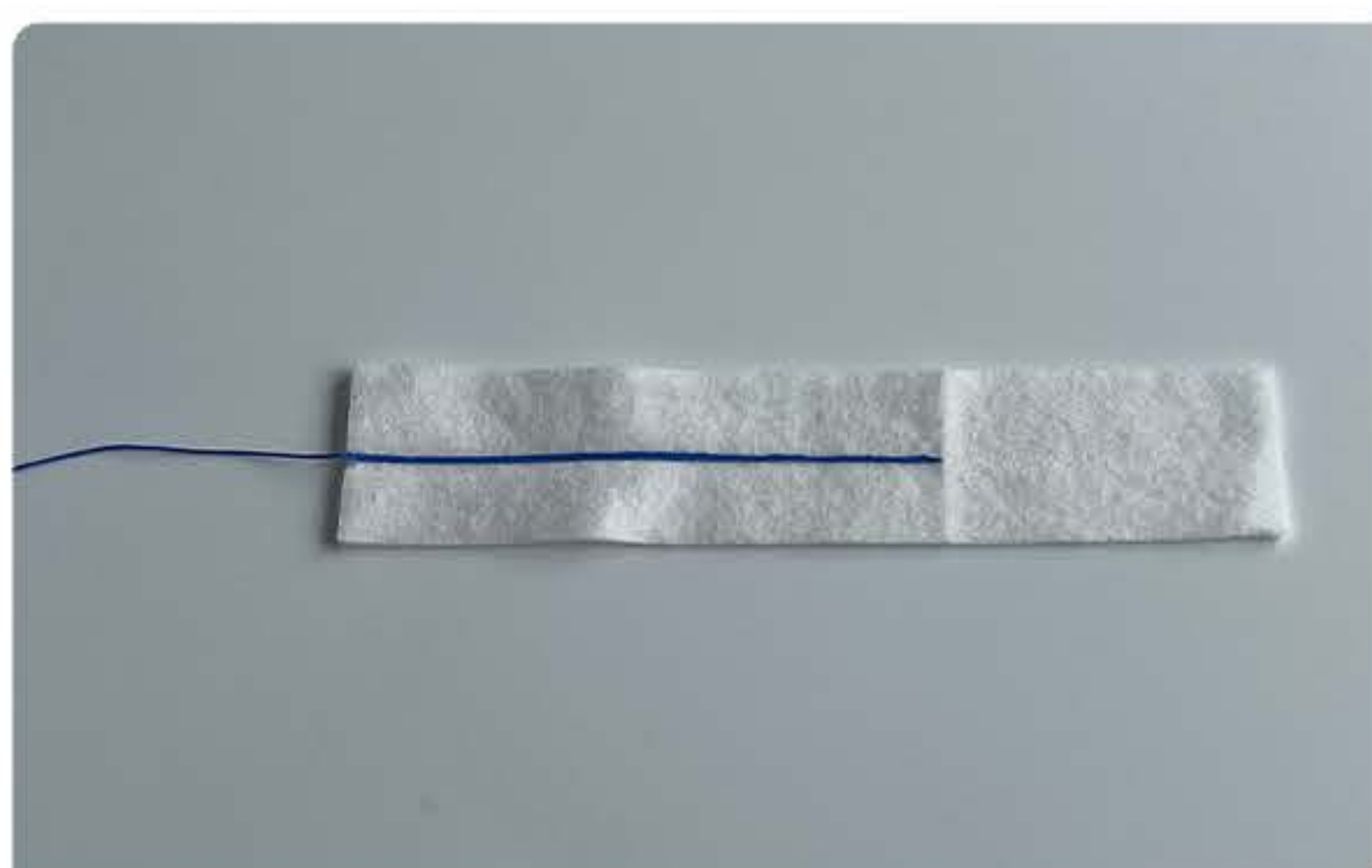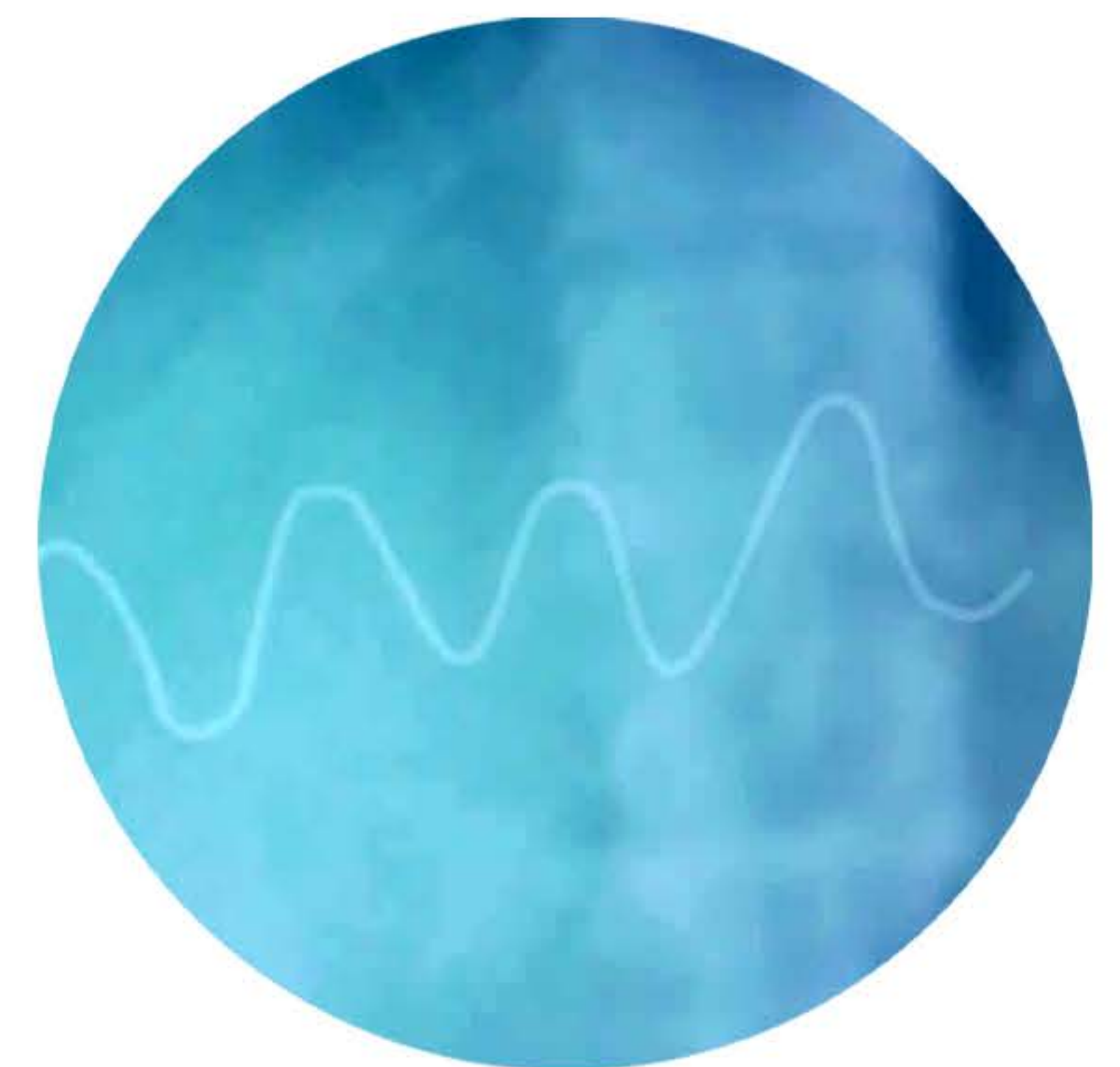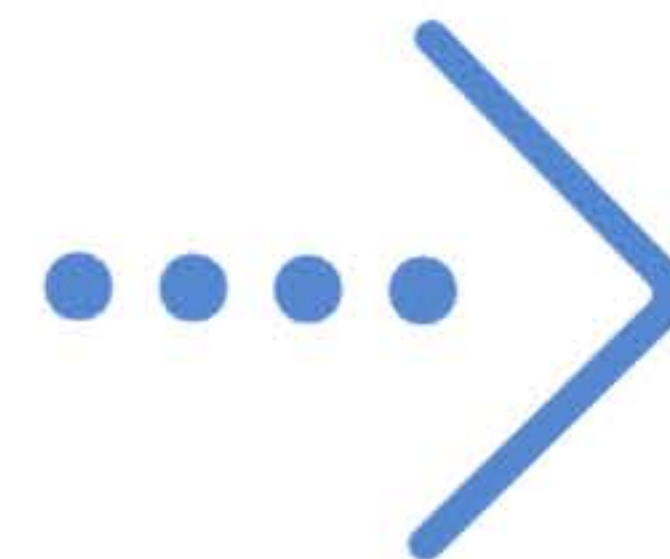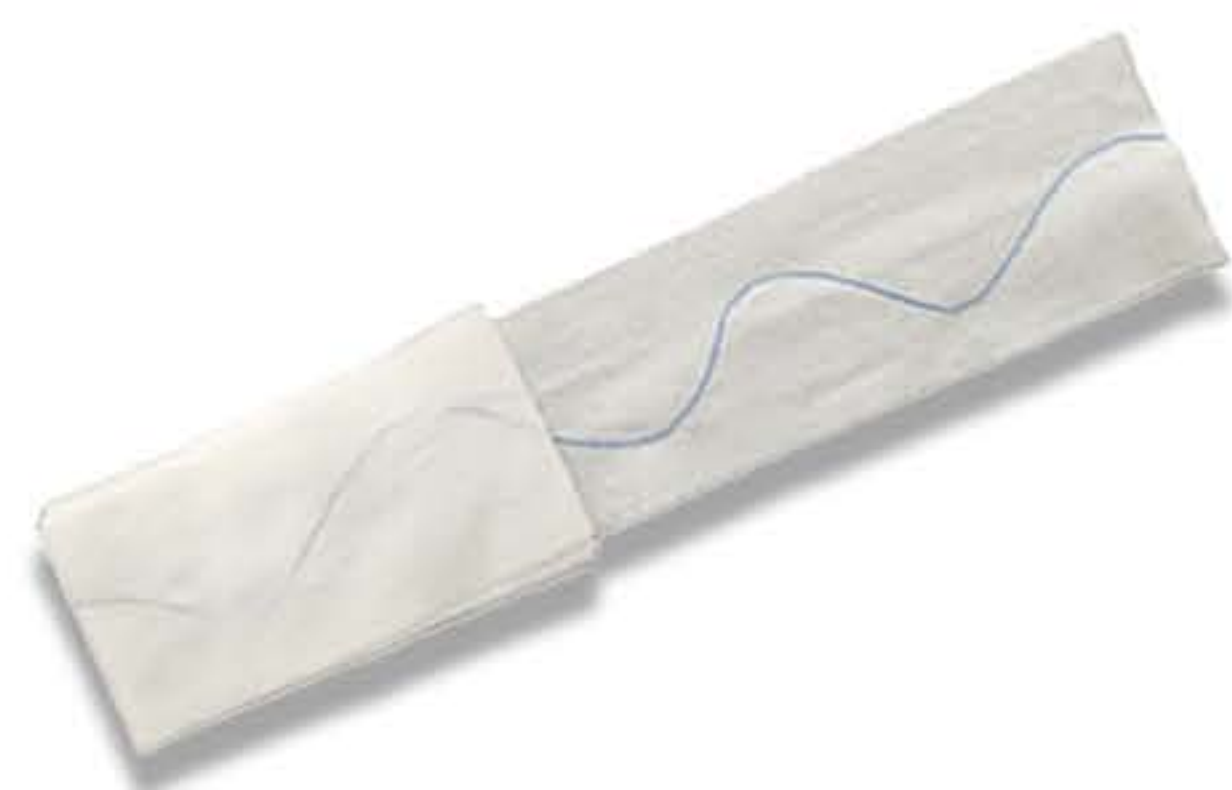

No	Size	Packing
1	10x10mm	10pcs/bag;200pcs/box
2	10x20mm	10pcs/bag;200pcs/box
3	10x40mm	10pcs/bag;200pcs/box
4	15x75mm	10pcs/bag;200pcs/box
5	20x20mm	10pcs/bag;200pcs/box
6	20x40mm	10pcs/bag;200pcs/box
7	25x25mm	10pcs/bag;200pcs/box
8	25x75mm	10pcs/bag;200pcs/box
9	40x40mm	10pcs/bag;20pcs/box
10	60x60mm	10pcs/bag;20pcs/box



森 工  
生 物

地址:江苏省南京市江北新区科创大道9号201室  
联系电话:02558355292  
网址:[www.purnote.com](http://www.purnote.com)  
邮箱:[sales@purnote.com](mailto:sales@purnote.com)

**NAN JING  
SENGONG**  
BIOTECHNOLOGY COMPANY LTD

ADDRESS: B8-201, 9 KECHUANG AVENUE,  
JIANGBEI NEW AREA,  
NANJING CITY,  
210000, CHINA  
TELEPHONE: +86 02558355292

