

Silicone Island Dressing

无纺布硅胶敷料




Product Description --产品介绍

- Traditional dressings use harsh adhesives which can damage or tear skin during dressing removal. Silicone nonwoven wound dressing is updated with silicone glue and solve this problem.

传统敷料采用的普通黏胶,在移除敷料时候会造成皮肤二次伤害,无纺布硅胶敷料升级采用硅胶有效解决该问题。

- Protect fragile skin with Silicone Island Dressing, a premium cover dressing with a specially-formulated silicone contact layer. This dressing protects new tissue and intact skin during dressing changes to support comfortable healing.

硅胶保护层,有效保护脆弱肌肤。促进伤口愈合。

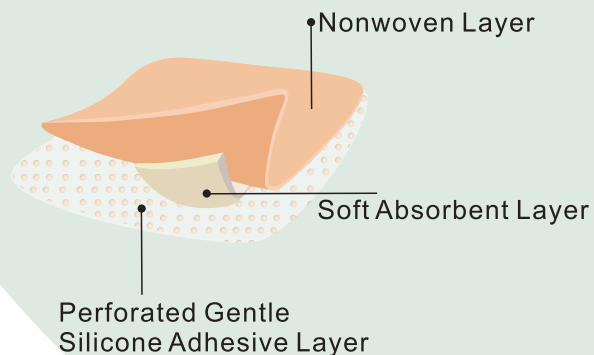

Dressing Features --特点

- Non-woven island dressing with an absorbent pad and silicone contact layer protects the wound site without sticking to granulating tissue

敷芯和硅胶保护层,可以有效的保护伤口避免撕裂伤。

- A gentle product for those at risk of skin tears with water-resistant backing and gentle adhesive border.

拒水层和硅胶有效避免皮肤撕裂。

- Low profile design increases flexibility and comfort

设计简单,患者使用灵活舒适。



Tel:86-20-85295649 Fax:86-20-85295797 <http://www.yafho.com> E-mail:info@yafho.com

Yafho 
WOUND CARE

单张尺寸: 205X280mm

材 料: 200克铜版纸

正面

Instructions for Use--使用说明

- Clean the wound in accordance with normal procedures.
按要求清洁伤口。
- Select the appropriate dressing size.
选用合适尺寸敷料。
- Prepare and clean the skin surrounding the wound area by removing excess moisture. Any excess hair should be clipped to ensure close approximation to the wound.
清洁伤口周围皮肤，保持干燥。必要时祛除周遭毛发确保有效贴合。
- Start to remove the protector material from the dressing and anchor the adhesive side of the dressing to the skin.
移除敷料保护层，将敷料粘贴区域对准需要贴的位置。
- Smooth the dressing over the wound removing the remaining protector material and ensure that the dressing is adhered all around the wound.
抚平敷料，移除余下的保护层，确保敷料有效覆盖伤口区域。
- Dressing Removal Lift one corner of the dressing and slowly peel back until completely removed from the wound.
按照敷料一角轻轻提拉，直至将整个敷料揭开。

Ordering Information--订货信息

Item 货品	Ref. 型号	Size(cm) 尺寸	Package 包装方式
Silicone Island Dressing 无纺布硅胶敷料	616002	2x4	10pcs/box
	616003	2x6	10pcs/box
	616338(round)	3.8x3.8	10pcs/box
	616022	2.5x7.2	10pcs/box
	616004	4x8.6	10pcs/box
	616048	5x7	10pcs/box
	616106	6x7	10pcs/box
	616108	8x10	10pcs/box
	616110	9x10	10pcs/box
	616115	9x15	10pcs/box
Other sizes are available 其他规格可定制			



Manufacturer: Yafho Bio-Technology Co.,Ltd
 生产商：台山市弘毅医疗用品有限公司
 Add: No. 12, 9Rd, Changlong Ind. Zone, Taishan city, Guangdong, China
 地址：广东江门台山市四九镇长龙工业区九路十二号之一
 Tel: 86-020-85295649 Fax: 86-020-85295797
 电话：86-20-85295649 传真：86-20-85295797
 www.yafho.com

Tel:86-20-85295649 Fax:86-20-85295797 http://www.yafho.com E-mail:info@yafho.com

Yafho 
 WOUND CARE